




**EXO-CAD**

# **Implant Library ADD Manual**

---

• **DIGITAL DEPARTMENT**

## EXO-CAD Implant Library ADD

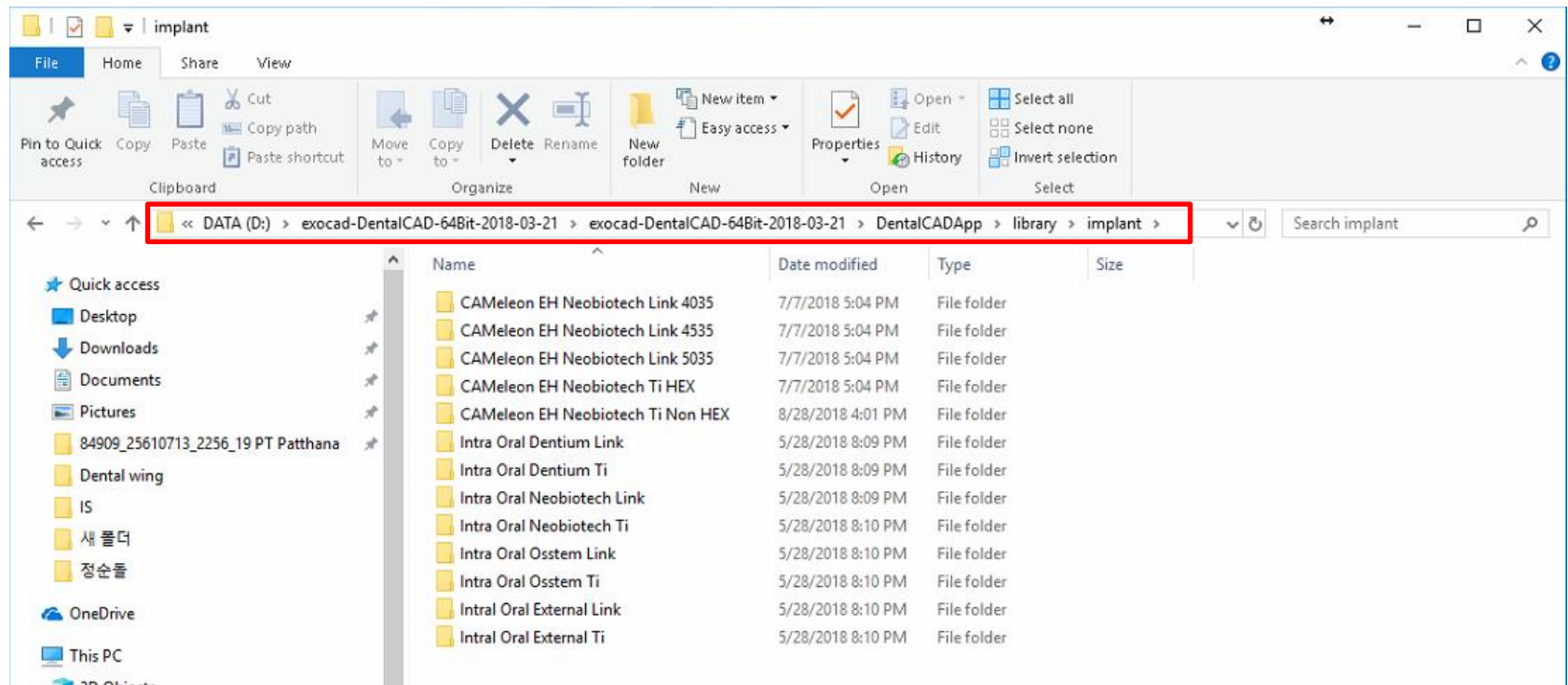

Enter the EXOCAD folder "DentalCADApp\library\implant" on the windows explorer


## EXO-CAD Implant Library ADD

The screenshot shows a Windows File Explorer window with the address bar path: `DATA (D:) > exocad-DentalCAD-64Bit-2018-03-21 > exocad-DentalCAD-64Bit-2018-03-21 > DentalCADApp > library > implant >`. The main pane displays a list of folders:

Name	Date modified	Type	Size
CAMEleon EH Neobiotech Link 4035	7/7/2018 5:04 PM	File folder	
CAMEleon EH Neobiotech Link 4535	7/7/2018 5:04 PM	File folder	
CAMEleon EH Neobiotech Link 5035	7/7/2018 5:04 PM	File folder	
CAMEleon EH Neobiotech Ti HEX	7/7/2018 5:04 PM	File folder	
CAMEleon EH Neobiotech Ti Non HEX	8/28/2018 4:01 PM	File folder	
Intra Oral Dentium Link	5/28/2018 8:09 PM	File folder	
Intra Oral Dentium Ti	5/28/2018 8:09 PM	File folder	
Intra Oral Neobiotech Link	5/28/2018 8:09 PM	File folder	
Intra Oral Neobiotech Ti	5/28/2018 8:10 PM	File folder	
Intra Oral Osstem Link	5/28/2018 8:10 PM	File folder	
Intra Oral Osstem Ti	5/28/2018 8:10 PM	File folder	
Intra Oral External Link	5/28/2018 8:10 PM	File folder	
Intra Oral External Ti	5/28/2018 8:10 PM	File folder	

A red box highlights the address bar path. A red arrow points from a folder icon labeled 'IntraOral\_Neobiotech\_IS' (also in a red box) to the 'implant' folder in the list. The text 'Copy (drag)' is written next to the arrow.

Copy(or drag) the library files to the "Implant" folder

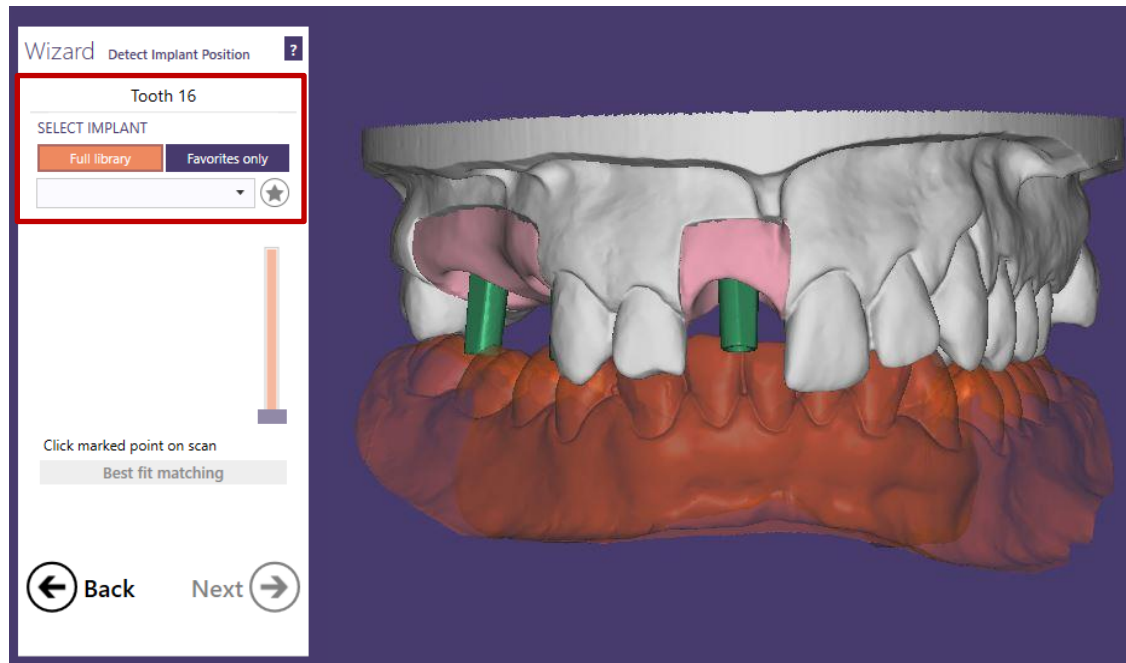

Click the "Select Implant" on EXOCAD.  
Then check the added library and select it.

1

## UNOFFICIAL EXTENSION OF IMPLANT LIBRARY

This implant type was not tested by the manufacturer of this software and is not signed.

Correct operation of the software with this data cannot be assured.

Type: CAMEleon Intra Oral NeoBiotech Ti

WARNING - UNOFFICIAL EXTENSION OF IMPLANT LIBRARY

Are you sure you want to use this implant type?



**Do not use**

Reject type and select a different one.



**Use anyway**

Malfunction of the software or inaccuracies may result.

If the warning message occurs,  
Click the "Use anyway" button and close.  
Then you can use new added library.

**Thank you**

A low-angle photograph of a modern, multi-story office building with a white facade and dark window frames. The building is viewed from a perspective that makes it appear to be leaning slightly to the right. The sky is a pale, overcast grey. The company name 'Neo Biotech' is printed vertically on the side of the building. The word 'Neo' is in a red, stylized font, and 'Biotech' is in a black, sans-serif font.

**Neo  
Biotech**